


Guidelines to view Echocardiograms and CathLab Images on MagicWeb. 7/22/2009

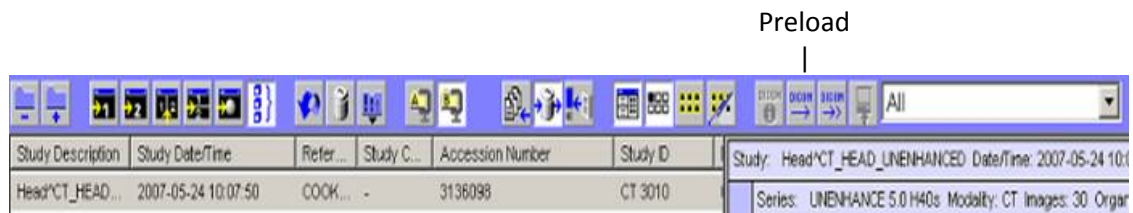

1. Have the Load All Series Button Pushed.. you will want to turn off this button, when looking at Chest Xrays, CTs, etc
2. Have the B Compression Button Pushed
3. Have the Use Local Cache Button Pushed
4. After you pick your exam to review, you want to click on the Preload Button
5. You should then be able to start looking at images in the study, by using the next Series buttons to move thru the study. The Images will continue to load, as you start looking at the images.
6. I've found that the Edge Enhance Button cleans up the images, especially the CathLab Images.
7. You can now Q/R images from the CathLab Server (ACOMNET) and the ProSolv Server (PROSOLVQR)
8. Having MagicWeb access to the ProSolv server, now allows you to look at not only MGH echoes, but also outlying hospital echoes. This also allows you to view images with MagicWeb from any adequate internet connection, using your URoam connection.