



# MagicWeb Upgrade to Version 50: Release Notes 7/18/2007

Here are the main tool enhancements with the Version 50 upgrade:

-  When you log onto MagicWeb, a patient list will not display until you've hit the **Start Data Query** button for your present settings or create a new search.
-  **Set Filter Properties.** The **Personal Settings** dialog box opens. You can define your own filters here. If you had previous filters created in the earlier version of MagicWeb and they do not perform as you want them to, please delete them and recreate new filters.
-  **Reset Filter Settings**
-  **Reset Input Fields**
-  **Start Data Query** and **Retrieve Study from DICOM Node.** Retrieve study from DICOM Node button allows you to select a number of studies to pull back at once, by holding the CTR key down while selecting studies and then hit the button.
-  **Forward** and **Backward** buttons for large patient lists
-  **Load to Basic MIP/MPR Viewer,** after selecting a series to load into 3D. Helpful hint: For PCs that don't have enough RAM to load a whole series, you may select desired thumbprints as below and load only those images into 3D.  

-  **Load all series as one,** is helpful for chest x-rays, plain films and cath lab images, where each images comes across as it's own series. Highlight the images you want to see, with this button pushed and load study. You then can you the next series button. **Note: you need to have this function turned off to load a specific series into a viewer.**
-  **Refresh** is helpful while big studies are crossing to MagicWeb, from either a modality or from another server. You don't have to go in and out of the viewer to refresh, you can just hit this button
-  **Query Report,** will bring over a copy of the NOVIUS report, if it's at a Final Status. **(THIS FEATURE IS ON HOLD UNTIL WE CAN DISPLAY THE STATUS OF THE REPORT FROM NOVIUS)**
-  **Use Local Cache.** If this button is activated, the images you download from the web server are stored in your local cache memory.
-  **Preload Study.** Loads all images of a selected study into the local cache for the duration of your session. When you subsequently load these images into a viewer and go to view them, the images are displayed immediately.
-  **Truncate Thumbnails for Large Series.** Toggles thumbnail display for large series: one or all thumbnails per series are shown in the image preview area.
-  **Select all preview** or **Deselect all previews** of thumbnails
-  **Status Information.** Display information about all data transfer jobs in the network
- The Insertion Date Column** may not be the same as the study date, as it represents the date that the images were sent to the MagicWeb server.
-  **Cancel Loading.** Stops Loading Images
-  **Report with Image Review,** after an image is select and the report is available.